



## USER MANUAL

### REAL-TIME PCR DETECTION KITS

#### General information

**Intended use:**

The **Real-Time PCR Detection Kits** are *in vitro* DNA tests, which are intended for the specific identification of various microorganisms in human biological samples. The kits are for research use only.

**Method:**

Real-time polymerase chain reaction; qualitative detection.

**DNA extraction:**

The "DNA-Technology" **PREP-NA**, **PREP-GS** and **PREP-RAPID DNA extraction Kits** are recommended. Some types of the samples must be pretreated (refer to the corresponding user manuals).



Use only **PREP-NA** and **PREP-GS Kits** for DNA extraction from biopsy samples, phlegm, bronchoalveolar lavage, gastric fluid and faeces.

We do not recommend **PREP-RAPID DNA Extraction Kit** for DNA extraction from male urogenital swabs.

**Features:**

PCR-Mix contains an internal control (IC). IC is intended for PCR quality and sufficiency of DNA assurance.

Positive control plasmid (C+) supplied with the kit is intended for specific PCR assessment.

We also recommend including in assay the negative control (C-) which is not supplied but very helpful for contamination control purposes. Use deionized water or sterile buffered saline instead of sample, starting from extraction step.

**Devices:**

The automatic analysis for DNA-Technology **Real-Time PCR Detection Kits** is available on "DNA-Technology" made DTlite<sup>1</sup>, DTprime<sup>2</sup>; the latest version of the software is available for download at <http://www.dna-technology.ru/eng/support/> or Bio-Rad Laboratories iCycler iQ thermal cyclers.



Please enquire DNA-Technology company's representative about compatibility of third-party Real-time instruments.

**Overall time needed to perform the analysis (excluding sample preparation procedure):**

from 1.5 h.

**The number of tests:**

48/96 (including one positive control and one negative control in each run).

#### Kits content

Reagent	Organoleptic parameters	Quantity	
		48 tests	96 tests
• Paraffin sealed PCR-mix	Colorless transparent liquid under white wax layer	48 tubes or 6 8-tube-strips (20 µL in each tube)	96 tubes or 12 8-tube-strips (20 µL in each tube)
• Taq-polymerase solution	Colorless transparent liquid	1 tube (total volume 500 µL)	2 tubes (500 µL in each tube)
• Mineral oil	Colorless transparent viscous oily liquid	1 tube (total volume 1.0 mL)	2 tubes (1.0 mL in each tube)
• Positive control	Colorless transparent liquid	1 tube (total volume 75 µL)	1 tube (total volume 150 µL)

#### Detection channels

Fam	Hex	Rox	Cy5	Cy5.5
Specific product	IC	-	-	-

<sup>1</sup> - 4S1, 4S2, 5S2, 6S1, 6S2 models.

<sup>2</sup> - 4M1, 4M3, 4M6, 5M1, 5M3, 5M6, 6M1, 6M3, 6M6 models.

## Samples

REF	Microorganism	Sample
R1-P002-23/4EU R1-P002-S3/4EU	<i>Bordetella pertussis</i>	Swabs from posterior pharyngeal wall
R1-P406-23/4EU R1-P406-S3/4EU	<i>Chlamydomphila pneumoniae</i>	Phlegm, bronchoalveolar lavage, swabs and washouts from nasal and oral pharynx
R1-P401-23/4EU R1-P401-S3/4EU	<i>Corynebacterium diphtheriae</i> - toxigenic strains	Swabs from fibrin filmoral pharynx, larynx and other mucous membranes, lesion fluid
R1-P204-23/9EU R1-P204-S3/9EU	Cytomegalovirus (CMV)	Saliva, urine, swabs from urethra, cervix, or posterolateral vaginal wall, peripheral blood mononuclear cells
R1-P501-23/4EU R1-P501-S3/4EU	<i>Helicobacter pylori</i>	Biopsy samples, gastric fluid and faeces
R1-P202-23/9EU R1-P202-S3/9EU	Human herpesvirus 6 (HHV6)	Peripheral lymphocytes, spinal fluid, saliva, urine etc.
R1-P203-23/9EU R1-P203-S3/9EU	Human herpesvirus 8 (HHV8)	Peripheral lymphocytes, sperm, prostate fluid, biopsy samples
R1-P322-23/9EU R1-P322-S3/9EU	Human papillomavirus 6 (HPV6)	Swabs from urethra, cervix, nasal and oral pharynx, biopsy samples etc.
R1-P323-23/9EU R1-P323-S3/9EU	Human papillomavirus 11 (HPV11)	
R1-P301-23/9EU R1-P301-S3/9EU	Human papillomavirus 16 (HPV16)	
R1-P301-23/9EU R1-P301-S3/9EU	Human papillomavirus 18 (HPV18)	
R1-P011-23/4EU R1-P011-S3/4EU	Human Parvovirus B19	Blood plasma, saliva, biopsy samples from heart valves, amniotic fluid, autopsy samples
R1-P003-23/4EU R1-P003-S3/4EU	<i>Listeria monocytogenes</i>	Spinal fluid, mucosal swabs, amniotic fluid, meconium, biopsy samples
R1-P411-23/4EU R1-P411-S3/4EU	<i>Mycoplasma pneumoniae</i>	Phlegm, bronchoalveolar lavage, swabs and washouts from nasal and oropharynx
R1-P012-23/4EU R1-P012-S3/4EU	<i>Streptococcus agalactiae</i>	Swabs from vagina, rectum, nasal pharynx, urine, spinal fluid, blood, autopsy samples
R1-P412-23/4EU R1-P412-S3/4EU	<i>Streptococcus pneumoniae</i>	Phlegm, bronchoalveolar lavage, swabs and washouts from nasal and oral pharynx
R1-P402-23/4EU R1-P402-S3/4EU	<i>Streptococcus pyogenes</i>	Swabs from urethra, cervix, posterolateral vaginal wall, lesions and mucosa of the amygdalae, posterior pharyngeal wall
R1-P001-23/9EU R1-P001-S3/9EU	<i>Toxoplasma gondii</i>	Spinal fluid, biopsy samples, etc
R1-P101-23/9EU R1-P101-S3/9EU	<i>Chlamydia trachomatis</i>	Swabs from urethra, cervix, or posterolateral vaginal wall

## Procedure

### 1 PCR amplification



The reagents and tubes should be kept away from direct sun light.

In case of using tubes in strips, strictly observe the completeness of the strips and caps to them. Do not use the caps to the strips of the other kits!

- 1.1 Mark tubes with paraffin sealed PCR-mix for test samples, negative control (C-) and positive control (C+).  
**Example.** If you need to test 5 samples, mark 5 tubes for samples, 1 tube for "C-" and 1 tube for "C+". Total number of tubes - 7.
- 1.2 Mix the Taq-polymerase solution thoroughly (3-5 s), then spin briefly (1-3 s).
- 1.3 Add 10 µL of Taq-polymerase solution into each tube. Avoid paraffin layer break.
- 1.4 Add one drop (~20 µL) of mineral oil into each tube. Close tubes.
- 1.5 Vortex the tubes with samples, "C-" and "C+" for 3-5 s and spin down drops for 1-3 s.



1. In case of using **PREP-GS DNA extraction kit**. After vortexing centrifuge the tubes with the DNA preparation at 16 000 x g for one minute at room temperature (from 18 °C to 25 °C) to precipitate the sorbent. If, after isolation, the supernatant containing the isolated DNA was transferred to new tubes, centrifugation is carried out for 1-3 s in a microcentrifuge-vortex.

Relative centrifugal force (RCF or g) depends on rotation frequency and rotor radius. To establish if your centrifuge meets the requirements apply to the exploitation manual for centrifuge.

2. Open the tube, add DNA sample (or control sample), then close the tube before proceeding to the next DNA sample to prevent contamination. In case of using tubes in strips, close the strip before proceeding to the next DNA sample to prevent contamination. Use filter tips.

- 1.6** Add 5.0  $\mu$ L of DNA sample into corresponding PCR-tubes. Avoid paraffin layer break. Use filter tips. Do not add DNA into the "C-", "C+" tubes.
- 1.7** Add 5.0  $\mu$ L of negative control (C-) which passed whole DNA extraction procedures and positive control (C+) into corresponding tubes. Avoid paraffin layer break.
- 1.8** Spin the tubes for 1–3 s.
- 1.9** Set the tubes to Real-time PCR Thermal Cycler..
- 1.10** For DTLite and DTprime thermal cyclers:  
Launch RealTime\_PCR software and choose the "Device operation" mode. Create and save new test if you do this test for the first time. In subsequent runs add the saved test to the protocol, specify the number and ID's of the samples, specify the position of the tubes in the thermal unit (1.9) and run PCR (see table 1).  
For iCycler iQ thermal cyclers:  
Switch on the thermal and optical units of the device and let it warm up for 30 min. Launch iCycler (or Bio-Rad iQ5) software. Create and save new protocol if you do this protocol for the first time. In subsequent runs add the saved protocol, configure the plate (create the file with the data on samples ID's and position) and run PCR considering the total PCR reaction volume equal to 35  $\mu$ L (see tables 2, 3).

**2 Data collection and data analysis** are operated by software and held in automatic mode.

Table 1. The PCR program for DTLite andDTprime thermal cyclers

Step	Temperature, °C	Min.	Sec.	Number of cycles	Optical measurement	Type of the step
1	80.0	0	30	1		Cycle
	94.0	1	30			
2	94.0	0	30	5		Cycle
	64.0	0	15		v	
3	94.0	0	10	45		Cycle
	64.0	0	15		v	
4	94.0	0	5	1		Cycle
5	10.0	...	...	...		Holding

Table 2. The PCR program for iCycler iQ5 thermal cyclers (with persistent well factor)

Cycle	Repeats	Step	Dwell time	Setpoint, °C	PCR/Melt Data Acquisition
1	1	1	1 min	80.0	
		2	1 min 30 sec	94.0	
2	5	1	30 sec	94.0	
		2	45 sec	64.0	
3	45	1	10 sec	94.0	
		2	45 sec	64.0	Real Time
4	...	...	...	10.0	Storage

Table 3. The PCR program for iCycler iQ thermal cyclers (with dynamic well factor)

Cycle	Repeats	Step	Dwell time	Setpoint, °C	PCR/Melt Data Acquisition
dynamicwf.tmo program					
1	1	1	1 min	80.0	
		2	1 min 30 sec	94.0	
2	5	1	30 sec	94.0	
		2	45 sec	64.0	
3	2	1	30 sec	80.0	Real Time
		PCR program			
4	45	1	10 sec	94.0	
		2	45 sec	64.0	Real Time
5	...	...	...	10.0	Storage

## Storage, shipping and handling requirements

All components of the kits must be stored at from 2 °C to 8 °C during the storage period.



Tubes with paraffine sealed amplification mixture should be stored in a dark place at a temperature from 2 °C to 8 °C during the whole shelf life period.

Transportation can be held by all types of roofed transport at temperatures between 2 °C and 8 °C during all storage period of the kit.

Shelf life – 12 months since the date of production in compliance with all transportation, storage and operation conditions.

Contact our customer service department regarding quality issues with the kit:












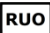

8 (800) 200.75.15 (toll-free call for Russia)

+7 (495) 640.16.93 (chargeable call for CIS and foreign countries).

E-mail: [hotline@dna-technology.ru](mailto:hotline@dna-technology.ru), [www.dna-technology.ru](http://www.dna-technology.ru).

Address: "DNA-Technology" LLC, 117587, Russia, Moscow, int. ter. Municipal District Chertanovo Severnoye, Varshavskoye shosse, 125 Zh, building 5, floor 1, office 12

### Key to symbols

	For research use only		Consult instructions for use
	Temperature limitation		Manufacturer
	Expiration date		Number of tests
	Date of manufacture		Catalogue number
	Single use		Do not expose to sunlight
	Caution		Not sterile
	Batch code		